

Patient Instructions

PLACE STICKER HERE

STEP 1: Sample Deposit

A. Lift toilet seat and position Saran Wrap across the rim of the toilet bowl. Make small dip to allow collection of stool sample. Lower the toilet seat




B. Make Bowel Movement onto Saran Wrap




STEP 2: Sample Collection


A. Unscrew the purple cap from the sample collection tube. **DO NOT POUR OUT THE LIQUID.**

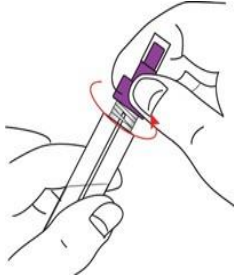

B. Poke spiral applicator into stool at 6 different sites. Use only enough fecal material to cover the tip of the applicator. **DO NOT CLUMP, SCOOP, OR FILL THE TUBE.**

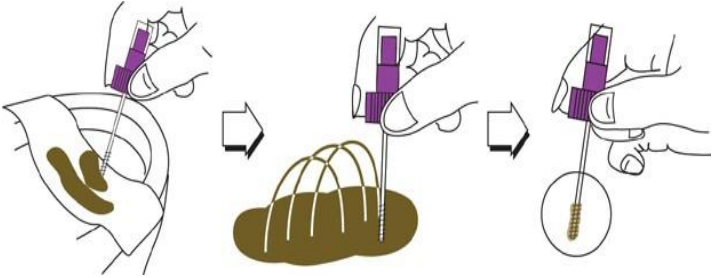


C. Screw the applicator back into the tube and secure tightly.

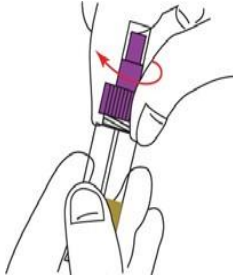

STEP 3: Sample Return

A. Write the collection date on top of this paper.



B. Insert sample collection tube into specimen bag. Insert paperwork into outside sleeve and return to your provider or AYUMETRIX.

